



CARE:

Wipe the inside of the wraps down with a damp cloth to clean and place back in the hard carry case for protection.

TIPS FOR USE:

While there are no cords for your horse to step on our get tangled in, we do recommend your horse be tied during treatment and supervised.

- Use prior to daily training sessions to loosen up tense muscles, tendons and joints
- Use prior to massage, acupuncture or other therapeutic techniques to stimulate and release trigger points
- Limit treatment time to 20-30 minute intervals 1 to 2 times per day
- Do not use over open wounds
- 3 light intensities to customize your need from maintenance use to injury recovery

4 Light mode settings gives you the diversity of setting your red light intensity based on your horse's needs from maintenance to recovery with time internal settings starting at 5 minutes up to 30 minutes. We recommend at least 20 minute treatments 2 times per day. The battery has an automatic time shut off.

L1 - 25% Light intensity - best for daily maintenance, or use on sensitive or bonier areas

L2 - 50% Light intensity - best for daily maintenance, or use on mild inflammation and soreness, can be used on sensitive or bonier areas as well as more muscled areas

L3 - 75% Light intensity - can be used for muscle recovery and therapeutic needs with mild to moderate inflammation and soreness. This is a more intense light with deeper penetration into the tissue.

L4 - 100 % Light intensity - Can be used for muscle recovery, deep tissue soreness and inflammation, excellent on fleshy and well muscled areas.

Technical Parameters:

- Power: 3W
- Input DC12V
- Wavelength Red 630nm and Blue 450nm
- Working Temperature - 30°-45°
- AC-DC Charger Battery Pack
- Input Voltage AC100-240V/AC200-240V
- DC11.4V 2200mAH
- Output Voltage DC12.6V 500mA

Warranty: 1 Year warranty on manufacturing defects included on LED wrap and cords. Horse damage (biting the wraps or cords or stepping on them) is not covered.



Cautions:

1. We do recommend you seek Veterinary care in case of an injury
2. Please do not clean the product in water
3. Please do not burn the battery to dispose of it
4. Limit the use of the products to no more than 30 minutes at one time.
5. Never directly look into the LED bulbs for long periods of time.



EQUICARE LED THERAPY® PURPLE LIGHT

How EQUICARE LED THERAPY® RED LIGHT Works



EquiCare LED Therapy® Purple Light stimulates a response from the body to help heal itself.

Specifically, 2 wavelengths, red light (630nm) and blue light (450nm) that combine to make purple that works to penetrate the levels of tissue to positively affect the tendons, ligaments, muscles and joints.

The process dramatically increases the blood flow sending more oxygen throughout the body, increasing collagen, endorphins, and capillary production, speeding up the removal of waste products, and helping to heal the entire body.

EquiCare LED Therapy® Purple Light can aid in treating arthritis, laminitis, navicular, tendon strains, muscle swelling and tightness. Multiple size LED Therapy wraps can be used to treat virtually any part of the horses body with the addition of the extension belt. (Neck, Back, SI, Stifles, cannon bones, knees, feet, hocks)

EQUICARE LED THERAPY® is a remarkable line of wraps at unbelievable prices for the everyday for the everyday horseman, trainer, therapist and vet.

EquiCare LED Therapy® Purple Light Benefits:

- ◆ Alleviation of pain
- ◆ Alleviation of inflammation
- ◆ Stimulates the immune system
- ◆ Deep Wound Healing
- ◆ Tissue regeneration
- ◆ Faster treatment of skin conditions: eczema, hair loss, scrapes, scratches, mud fever, cuts and wounds
- ◆ Effective treatment of hoof issues: laminitis, navicular, abscesses, and ringbone
- ◆ Effective treatment of sore backs, shoulders, strains, splints, stifle problems, tendons, and ligaments
- ◆ Safer alternative to laser light therapy with no side effects
- ◆ Non-ablative process
- ◆ Fast treatment time
- ◆ Easy to use Wraps for all applications
- ◆ Battery charged- no cords to trip over
- ◆ A dual benefit kit - the red light penetrates up to 2 1/2" deep speeding up tissue regeneration, while the blue light treats dermal layer bacteria



Kits Include: (2) Medium LED Wraps, (2) Large LED Wraps (2) LED Bell Boots, (6) Batteries, (6) Chargers, (2) Extension Belts all in an Aluminum Case

