

Schneiders®

Horses define who we are



EQUICARE LED THERAPY® Red Light

Human Shoulder Wrap



LED light therapy stimulates a response from the body to help heal itself. Specifically, 2 types of wavelengths, a visible red light (660nm) and an invisible near infrared light (850nm) work in unison to penetrate the levels of tissue to positively affect the tendons, ligaments, muscles and joints. The process dramatically increases the blood flow sending more oxygen throughout the body, increasing collagen, endorphins, and capillary production, speeding up the removal of waste products, and helping to heal the entire body. EquiCare LED Therapy® can aid in treating arthritis, sprains, tendon strains, muscle swelling and tightness.

56 LED Lights

Wrap is 9" tall with 3 extension belts lengths to fit most

EquiCare LED Benefits:

- Alleviation of Pain
- Alleviation of Inflammation
- Stimulates the immune system
- Tissue Regeneration
- Effective treatment reduces pain from Arthritis, Tendinitis, Rotator cuff injury, Bursitis, and Collar bone injury
- Safer alternative to laser light therapy with no side effects
- Non-ablative process
- Fast treatment time
- Easy to use Wraps for all applications
- Battery charged - no cords to trip over.

Tips for Use:

- Use daily to loosen up tense muscles, tendons and joints
- Use prior to massage, acupuncture or other therapeutic techniques to stimulate and release trigger points
- Limit treatment time to 20-30 minute intervals 1 to 2 times per day
- Do not use over open wounds

4 Light mode settings gives you the diversity of setting your red light intensity based on your horse's needs from maintenance to recovery with time internal settings starting at 5 minutes up to 30 minutes. We recommend at least 20 minute treatments 2 times per day. The battery has an automatic time shut off.

L1 - 25% Light intensity - best for daily maintenance, or use on sensitive or bonier areas

L2 - 50% Light intensity - best for daily maintenance, or use on mild inflammation and soreness, can be used on sensitive or bonier areas as well as more muscled areas

L3 - 75% Light intensity - can be used for muscle recovery and therapeutic needs with mild to moderate inflammation and soreness. This is a more intense light with deeper penetration into the tissue.

L4 - 100 % Light intensity - Can be used for muscle recovery, deep tissue soreness and inflammation, excellent on fleshy and well muscled areas.

Care:

Wipe the inside of the wraps down with a damp cloth to clean and place back in the hard carry case for protection.

Cautions:

1. Please see your Doctor if pain is persistent
2. Please do not clean the product in water
3. Please do not burn the battery to dispose of it
4. Limit the use of the products to no more than 30 minutes at one time.
5. Never directly look into the LED bulbs for long periods of time.

Technical Parameters:

- Power: 3W
- Input DC12V
- Wavelength 660nm/ 660nm+850nm/
- Working Temperature -32 degrees to 113 F
- AC-DC Charger Battery Pack
- Input Voltage AC100-240V/AC200-240V
- DC11.4V 2200mAH
- Output Voltage DC12.6V 500mA Warranty: **1 Year warranty on manufacturing defects included on LED wrap and cords.**



Kit Includes:

- Carrying case
- LED Wrap,
- Battery & Charger
- 3 Extension Belts

Schneiders[®]
Horses define who we are[™]

800.365.1311 | sstack.com

8255 E. Washington Street
Chagrin Falls, OH 44023