

10. To adjust pressure lift up on nob. Turn clockwise to increase. Counterclockwise to lower. Then press nob back down to lock it.



11. Proceed to Protocols



HELPFUL HINTS

To help your horse get use to nebulizing, you can hold your hand over the large port for just a minute to help them get used to it. Once they take a deep breath remove your hand, do this several times to teach them how to breathe in.

We highly suggest changing your med cups and tubing with every bottle of silver as they do wear out. If silver builds up rinse with water.

**NEVER USE PURE IN CUPS
MASK ONLY**

EQUINE RESPIRATORY THERAPY PROTOCOL WITH SILVAPLEX. WE SUGGEST STARTING WITH 8 STRAIGHT DAYS TO CLEAN RESPIRATORY SYSTEM OUT.

8-Day Treatment (*Bleeders –Lung Infections
- Chondritis/Chondromas*)

8 straight days 15cc 25 minutes a day then assess from there. If trying to get off Lasix, drop down ½ cc of Lasix each run until they can compete without it, or you feel comfortable with a lower dose. Call if you have any questions, 405-317-3396

3-Day Treatment (*Pre-race –Upper/lower
bacterial or viral infections*)

Once a day for 3 days 15cc Silvaplex 25-30 minutes.

Single Treatment (*Allergies –Coughs –Pre-
race –Race Day*)

15cc of Silvaplex 25-30 minutes. If using day of competition, do treatment at least 2 hours out so they can exudate anything before competing.

* If conditions continue contact your Veterinarian.
****List of medications that can be nebulized can be found on our website: www.equiresp.com

ALWAYS CLEAN MASK WITH PURE AFTER USE. SPRAY ALL SIDES OF MASK INSIDE, TURN UPSIDE DOWN TO GET INTO GASKET. LET SET ONE MINUTE THEN WASH OUT WITH WATER. NEVER USE CLOROX OR HARSH CHEMICALS. The mask may split a little on each side, this is normal to ensure a fit on most size horses.




405-317-3396 | equiresp@pldi.net

www.equi-resp.com



ELITE INSTRUCTIONS & PROTOCOLS

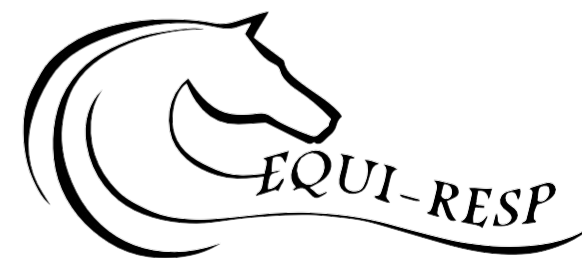

Respiratory
Products for the
Equine Market

Read the Intended Use and Safety Information before using. Damage to unit revokes warranty.

Any damage in shipping must be reported within 24 hours of receipt of equipment.

INTENDED USE

Equi-Resp Elite is a portable equine nebulizer designed specifically for animal use. It is intended for use with veterinarian prescribed drugs or SilvaPlex. The end user is liable for the safe operation of the Equi-Resp Elite and its parts as outlined in the instructions for use. Medications must always be prescribed by a veterinarian. SilvaPlex is NOT a drug and safe to use.

SAFETY INFORMATION


When using the Equi-Resp Elite for the first time remember it is the first time for the horse too, so caution is advised. Softly place the mask over the horses' nose and mouth with the small port position between the nostrils. If the horse is resisting the mask, be patient and let them look at it and smell it before attempting again. If the horse gets scared during a treatment, the quick release buckle will allow you to quickly drop the mask off the horse.

INSTRUCTIONS FOR USE

1. Make sure you remove Styrofoam from unit.
2. Set compressor outside the stall. Do not turn on yet.



3. First time use, leave horse in stall for safety.
4. Set unit up completely before putting mask on horse.
5. Attach tubing to unit with metal hose clamp. **HOSE CLAMP MUST BE USED**

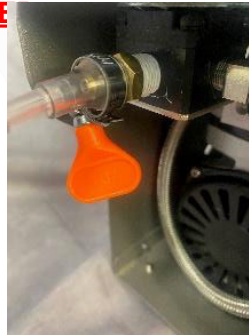

6. Fill Med Cup to 15cc. If you overfill it will not work.



7. Put mask on horse with a snug fit. Position the port between the nostrils about 3-4 inches away from nose.



8. Attach tubing to mask



9. Turn unit on, check pressure gauge. Weather and altitude can change pressure. Should be between 35-40 PSI for optimum use.



TURN PAGE OVER FOR REST OF INSTRUCTIONS