

Read the Intended Use, Safety Information before using. Damage to unit revokes Warranty. Any damage in Shipping must be reported Within 24 hours of receipt of equipment

Intended Use

Equi-Resp EquiMist is a portable equine nebulizer designed specifically for animal use. It is intended for use with veterinarian prescribed drugs or SilvaPlex . The end user is liable for the safe operation of the EquiMist and its parts as outlined in the instructions for use. Medications must always be prescribed by a veterinarian. SilvaPlex is NOT a drug and safe to use.

Safety Information

When using the EquiMist for the first time remember it is the first time for the horse too, so caution is advised. Softly place the mask over the horses' nose and mouth with the small port position between the nostrils. If the horse is resisting the mask, be patient and let them look at it and smell it before attempting again. If the horse gets scared during a treatment, the quick release buckle will allow you to quickly drop the mask off the horse.

INSTRUCTIONS FOR USE

READ ALL THE INSTRUCTIONS

1. Charge overnight first time with med cup on. When charged **green** light will blink slowly.



2. Always fill cup **OFF** the unit to keep liquid from leaking into controller.



3. If using pharmaceuticals, it is best to have one cup for meds and one for SilvaPlex.
4. Slide med cup back in grove till it clicks. If not properly seeded it will blink red/orange.



4. Unit can run for 20 minutes then shut itself off or when cup is empty it will make a shrill whistle sound.

5. Always empty the med cup as any product Left in the cup can deteriorate the mesh.
6. Clean med cup after each use. Fill with distilled water and run for 15 seconds, discard any left in cup.

7. DO NOT TOUCH THE MESH OR TRY TO CLEAN WITH Q TIP OR ANYTHING ELSE.

8. Med Cups are designed to be replaced. If they get too dirty, they simply cannot do their job.



9. Secure the mask and ensure horse is comfortable. Position the small port between the nostrils and 3-4 inches away from nose.
- 10.. Press Blue Button to turn nebulizer on. Never use sharp object, use ball of thumb placed over complete button.



TURN PAGE OVER FOR REST OF INSTRUCTIONS

Once you have completed nebulizing the horse, take mask off and clean with Pure.

11. When unit needs charging the **green** light turns to **red**.
12. Recharge should take less than 5 minutes.

EQUIMIST INDICATOR LIGHTS

Power On	Solid green light
Power Off	No Light
Low Battery	Solid Red Light
Battery Depleted	Solid red for 3 seconds then goes off
Medicine Cup Not Attached	Blinking Orange/Red Light
Battery Charger Plugged in and not Charging	No light
Battery Charger Plugged in and unit Turned on	Blinking green light
Battery fully charged	Slow blinking green light

Equine Respiratory Therapy Protocol with SilvaPlex. We suggest starting with 8 straight days to clean respiratory system out.

8-Day Treatment

8 straight days 6cc cup once a day.
Single Treatment (Allergies – Coughs – Pre-race – Race Day 6cc of SilvaPlex 2 hours from competition.

3-Day Treatment (Pre-race – Upper/lower bacterial or viral infections)
Once a day for 3 days 6 cc SilvaPlex.

BLEEDERS AND SERIOUS ISSUES

The Elite system is proven to help with these issues. We do not suggest the Equi-Mist for these as it was designed as an accompaniment to the Elite and only reaches the upper respiratory system.

*** If conditions continue contact your Veterinarian.**

******List of medications that can be nebulized can be found on our website:**

www.equiresp.com

ALWAYS CLEAN MASK WITH PURE AFTER USE. SPRAY ALL SIDES OF MASK INSIDE, TURN UPSIDE DOWN TO GET INTO GASKET. LET SET ONE MINUTE THEN WASH OUT WITH WATER. NEVER USE CLOROX OR HARSH CHEMICALS.


The mask may split a little on each side, this is normal to ensure a fit on most size horses.

NEVER USE PURE IN CUPS ONLY MASK



EQUI-MIST INSTRUCTIONS AND PROTOCOLS

**Video of instructions also on
webpage**

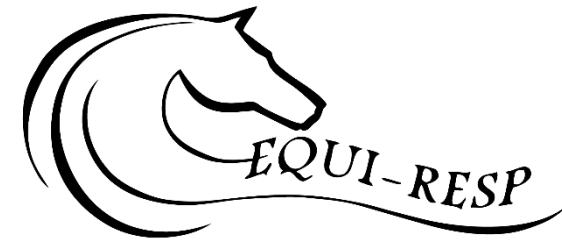

www.equi-resp.com
equiresp@pldi.net
405-317-3396